



PAD A CHEEK®

Mask Liner Amara View



To install the **PAD A CHEEK® - Amara View Mask Liner** on your mask:

1. Remove the top and bottom straps from the mask.
2. Lay the liner out like Photo 1.
3. Place the top of the mask into the liner with the label at the top of the seal of your mask like Photo 2.
4. Pull the sides of the mask liner over the places where the top straps attach like in Photo 3.
5. The side where the hose attaches should now look like Photo 4.
6. Pull the legs up and secure with the Velcro as in Photo 5.
- Make certain no exhalation vents are covered with the Mask Liner.**
7. Smooth the **PAD A CHEEK® - Mask Liner** over the seal.
8. The mask should look like photo 6. Attach the straps to your mask. You can attach the straps before putting the legs up if you wish.

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo 1



Photo 2



Photo 3



Photo 4



Photo 5



Photo 6

The **PAD A CHEEK® Mask Liner** is made from 100% polyester (micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK® 434-985-4003
157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com web site www.padacheek.com Designed and Sewn in the USA