


BPS Auto Charge DC Cable

Battery Power Solutions' Auto Charge DC Cable allows you to safely and efficiently charge the C-100 CPAP Battery Pack via the cigarette light/power socket of a vehicle.

INSTRUCTIONS

- Locate the connector tip labeled "N" in the Battery Pack Kit.
Each connector tip included in the C-100 Battery Pack Kit has a very small letter embossed on the top of the black plastic portion of the tip. Please note this letter can be difficult to read but it is present.
- Insert the "N" connector tip into the auto charge DC cable.
- Insert the auto charge DC cable with "N" connector tip attached into the DC in jack of the Battery Pack.
DUAL BATTERY PACK CHARGING:
 - Insert the auto charge DC cable with the "N" connector tip attached into the DC power pigtail cord.
 - Insert the DC power pigtail cord into the DC in jacks of both Battery Packs.
- Turn the vehicle on and insert the cigarette lighter plug end of the auto charge DC cable into the cigarette lighter/power socket of the vehicle to begin the charging process.
The vehicle must be running during the charging process to prevent draining the vehicle's battery and to ensure the Battery Pack receives proper voltage to successfully recharge.
- When charging is complete remove the auto charge DC cable from the cigarette lighter/power socket.

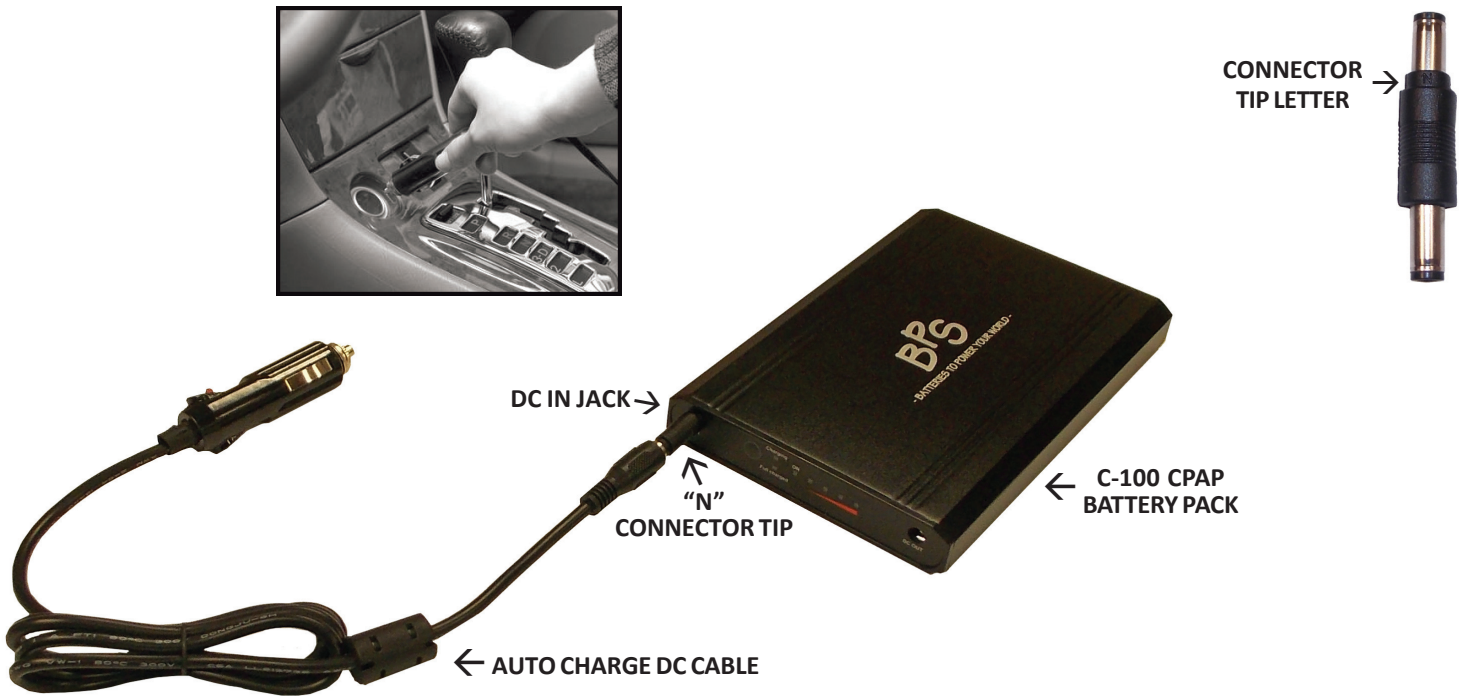

CHARGING NOTES: The typical recharge time when the Battery Pack is fully discharged is 4-5 hours (per battery); however, this may vary depending upon the vehicle's battery type. It is normal for the AC power supply and Battery Pack to be warm to the touch during the charging process.