RESMED

Fitting Guide



 Prior to fitting the mask, remove the elbow from the mask by pressing the side tabs.



• Connect the elbow to the device air tubing.



 With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



 Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



 Unfasten the Velcro[®], pull the upper headgear straps evenly until they are comfortable and firm, and reattach the Velcro.



• Repeat step 5 with the lower headgear straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the side tabs and pushing into mask frame.
- Turn on the airflow device.

Mirage[™] FX

www.resmedmiragefx.com



