


Filling the water tub

1. Slide the latch and lift open the flip lid.
2. Remove the water tub.
3. Fill the water tub (through the center hole) with distilled or deionized water up to the maximum water level mark (380 mL).
4. Return the water tub to the H5i.
5. Close the flip lid ensuring that it clicks into place.

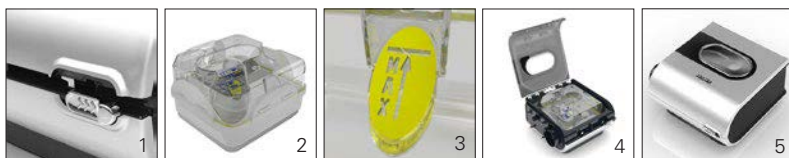

RESMED

AutoSet™ / AutoSet™ for Her / Elite™ / Escape Auto™ / Escape™

POSITIVE AIRWAY PRESSURE DEVICES

H5i™ / H5i™ for Her


HEATED HUMIDIFIER

Quick Setup Guide

English



Setup

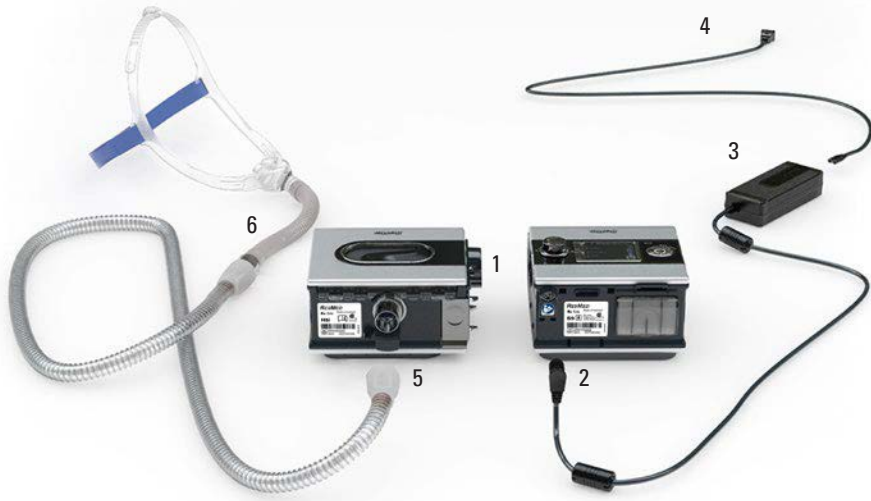

1. Align your H5i with your S9 and push them together until they click into place.
2. Connect the DC plug of the power supply unit to the rear of the S9.
3. Connect the power cord to the power supply unit.
4. Plug the other end of the power cord into the power outlet.
5. Connect one end of the air tubing firmly onto the air outlet.
6. Connect the assembled mask system to the free end of the air tubing.

Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on www.resmed.com on the Products page under Service & Support.

Adjusting humidity level

Ranging from OFF to 6, you can adjust the humidity level at any time to find the setting that is most comfortable for you. To adjust your level of humidity:



1. From your HOME screen, turn the dial until the HUMIDITY LEVEL screen is displayed in blue.
2. Press the dial. The selection is highlighted in orange.
3. Turn the dial until the humidity level that you require is displayed.
4. Press the dial to confirm your choice.

Adjusting Climate Control

This feature is available only if the ClimateLine™ heated air tubing is connected. You can adjust the air temperature to find the setting that is most comfortable for you.



1. From your HOME screen, turn the dial until the CLIMATE CTRL screen is displayed in blue.
2. Press the dial. The selection is highlighted in orange.
3. Turn the dial until the temperature that you require is displayed.
4. Press the dial to confirm your choice.

Adjusting ramp time

Designed to make the beginning of treatment more comfortable, ramp time is the period during which the pressure increases from a low start pressure to the treatment pressure. To adjust your ramp time:



1. From your HOME screen, turn the dial until the RAMP screen is displayed in blue.
2. Press the dial. The selection is highlighted in orange.
3. Turn the dial until the ramp time you require is displayed.
4. Press the dial to confirm your choice.

S9 Essentials

S9 Essentials is designed to make device interaction and menu navigation easier for you. If enabled by the clinician, S9 Essentials disables the Info and Setup functionality so that you can simply start and stop therapy and adjust ramp, humidification and Climate Control.